## ZAMST A1 (ANKLE SUPPORT) INSTRUCTIONS FOR USE

EN

Thank you for purchasing this product. For safety reasons, be sure to read this user manual before use. Also, store it in a location where you can refer to it at any time.

### Read this first:

- This product is side (left and right) specific. The left and right sides of the product are not interchangeable.
- Support may not be provided by this product under certain usage conditions.
- This product is not designed to treat ankle disorders.
- Check that the product size is correct before use.

### When using the product:

- Due to the characteristics of the materials used in this product, the color may change depending on storage conditions.
- When storing the product, avoid applying excessive weight to the hook-and-loop fastener or raised fabric. Otherwise, the hook-and-loop fastener may not work effectively.
- Do not wear the product while driving. Doing so may have an adverse effect on your driving ability.

### Precautions for use - Be sure to read this:

Safety precautions are divided into the following sections, according to the level of danger involved.

**⚠ WARNING:** Indicates a potentially hazardous situation, which may result in serious personal injuries if not avoided.

**⚠ CAUTION:** Indicates a potentially hazardous situation, which may result in minor personal injuries and/or property damage if not avoided.

### ⚠ WARNING

- To prevent abnormalities or worsening of symptoms, people who suffer from the following conditions should refrain from using the product:
  - Those predisposed to allergies or with sensitive skin.
    - \* This product contains natural rubber. Natural rubber contains latex allergens and may in certain rare cases cause allergic symptoms such as itching, redness, hives, swelling, fever, difficulty breathing, asthma-like symptoms, a drop in blood pressure, or shock. If any of the above-mentioned symptoms are observed, immediately stop using this product and seek medical attention.
  - Those who have a fracture, cut, numbness, swelling, eczema, rash or other abnormalities in the area where the product will be worn.
- If the product is not used correctly, it may cause circulatory problems or injury. Be sure to follow the warnings below and use the product correctly:
  - The product should only be used on ankles.
  - Be sure to use the correct size product.
  - Do not use the product while sleeping.
  - If wearing the product for a long period of time, readjust it as necessary. The application of constant pressure for a long period of time may cause skin and/or circulatory problems.
  - When using the product, be sure to put it on in the correct orientation (top/bottom, inside/outside, front/back).
  - Follow the attachment steps in order to use this product correctly.
  - Do not wrap the main body of the product or the straps too tightly. Doing so may cause skin and/or circulatory problems.
  - If the product loosens or comes off during use, be sure to refasten it correctly by following the noted steps from the beginning.
  - If the product shows any signs of wear or damage, or the hook-and-loop fastener no longer works well, do not use the product.
  - Do not attempt to modify or repair the product.
- The personal-fit-stays are made of plastic. To prevent personal injury, do not use the product in the following situation:
  - Where contact with the product may injure others.
- If any of the following abnormalities occur when using this product, immediately stop using it, and consult a physician. Continued use may worsen symptoms:
  - If this product causes discomfort.
  - If an injury, fracture, cut, pain, numbness, swelling, eczema, rashes, etc. occurs in the area where the product is worn.

### ⚠ CAUTION

- Hard materials are used for the hook-and-loop fastener. In certain rare instances, fingers or skin may be damaged, so use the product carefully.
- Be sure to wear the product over a sock. Do not wear it directly on your skin.
- Sweat and friction may cause the color of this product to fade or be transferred to other materials.
- Make sure the hook-and-loop fastener does not attach to clothing. If it does, clothing may become frayed or be damaged.

## Care and cleaning

### ⚠ CAUTION


- Do not wash this product with light colored items. By doing so, color may fade or be transferred to other materials.
- If washing the product together with clothing, etc., be sure to use a laundry net. If the hook-and-loop fastener becomes attached to clothing, it may cause the clothing to become frayed or be damaged.

- This product can be washed in a washing machine. After washing, hang it to dry in the shade.
- When washing this product, attach the hook-and-loop fastener to the main body. Attempting to wash this product without closing the hook-and-loop fastener may weaken its fastening capability.
- Do not use an iron or chlorine bleach, however, oxygen bleach can be used.
- Do not tumble dry. Doing so may damage the product.
- This product cannot be dry cleaned. Wet cleaning is allowed, however, avoid tumble drying.

# How to apply


- Before attaching this product, carefully read the "Precautions for use - Be sure to read this" section. Then follow the steps provided to ensure the product is correctly attached.
- The steps below use the right foot for reference. Follow the same steps for the left foot.

1. Unfasten all the hook-and-loop fasteners of the product. Confirm the top/bottom, inside/outside, and front/back of the product.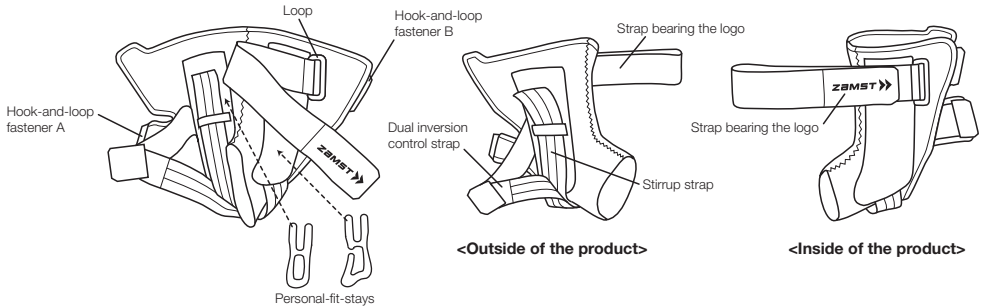


Fold back the upper strap with the ZAMST logo and temporarily fasten it to the main body to make the product easy to apply.

2. Position your ankle at right angle, open the rear entry of the product and slide your foot into the opening from the back.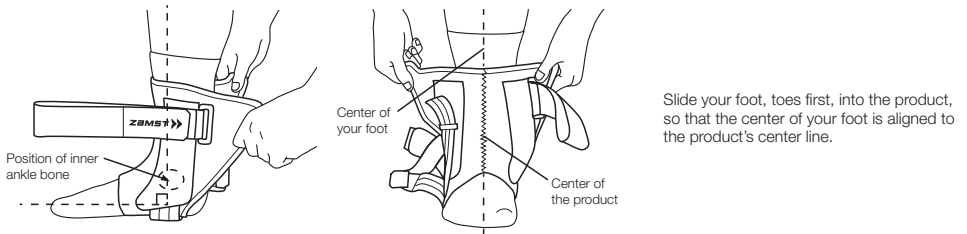

Slide your foot, toes first, into the product, so that the center of your foot is aligned to the product's center line.


3. Fasten hook-and-loop fasteners A and B.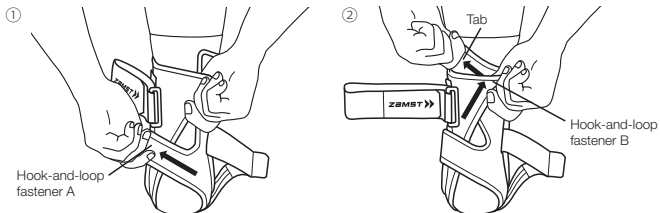

① Place your feet on the floor with your foot at right angles to your lower leg. Fasten hook-and-loop fastener A, pulling it diagonally upward.

② Hold hook-and-loop fastener B and the tab. Wrap the support around your ankle by pulling hook-and-loop fastener B diagonally upward and fastening it. Tightly wrap your ankle without creating wrinkles or sags.

4. Fasten the stirrup strap.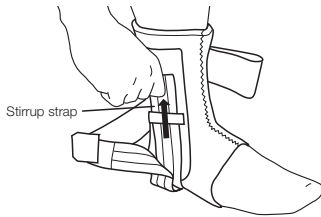


With your feet on the floor, pull the stirrup strap up to an extent that it does not sag, fasten the strap to the product, and apply firm pressure to the hook-and-loop fastener.

\* Do not pull the strap up with excessive force as doing so may cause skin and/or circulatory problems.

5. Pull the dual inversion control strap to fasten it to the product.


The effect of the dual inversion control strap varies depending on which direction you pull it.

① When the strap is pulled diagonally upward: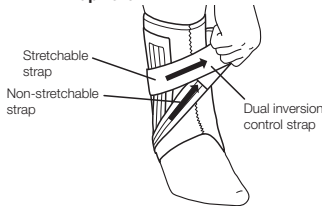

① When the strap is pulled diagonally upward: Inversion is restricted because the non-stretchable strap is contracted.


② When the strap is pulled laterally (inward):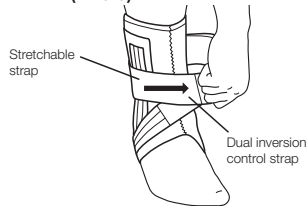

② When the strap is pulled laterally (inward): Side-to-side instability is restricted because the stretchable strap is strongly elongated.

6. Fasten the strap bearing with the logo.

Inner side view (big toe side)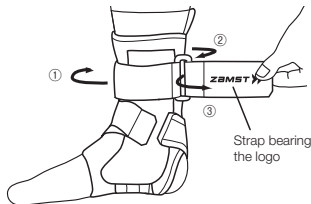

Wrap the strap bearing the logo around your ankle (①), pass it through the loop (②), fold it back, and fasten it (③).


\* Do not overtighten the straps as doing so may cut off blood circulation to the foot or lead to skin and/or circulatory problems.

7. Make sure that the product is not loose or out of place. If you feel any discomfort, repeat the steps from the beginning to reattach the product.

The product correctly fastened (right foot)

<Front view>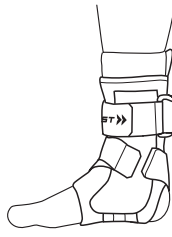

<Inner side view>

<Back view>

\* In the illustration, the logo is facing frontward. The logo does not always come into the center. Its position may differ according to the individual user.  
\* Adjust the strap bearing the logo as necessary to ensure a snug fit.

Disclaimer:

\* The product specifications and appearance are subject to change without notice for the purpose of improvement.

\* When disposing of this product, be sure to follow all disposal ordinances and regulations as specified by the local government.

\* We take all possible measures to ensure product quality, however, if you find any defects, etc. in our products, please contact Nippon Sigmax Co., Ltd.

"ZAMST" and "ZAMST logo" are trademarks or registered trademarks of Nippon Sigmax Co., Ltd in Japan, the United States and other countries.

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