ZAMST ICE BAG INSTRUCTIONS FOR USE

EN

Thank you for purchasing this product. For safety reasons, be sure to read this user manual before use. Also, store it in a location where you can refer to it at any time.

Usage:

- First-aid treatment of injury (cooling)
- Post-exercise cool-down

Read this first:

- Depending on the usage conditions, this product may not function as expected.

When using the product:

- Due to the characteristics of the materials used in this product, the color may change depending on storage conditions.
- Putting ice with sharp edges into the ice bag may damage the inner coating of the bag, causing water to leak out. Crush ice or add some water to smoothen any sharp edges before putting it into the bag.
- When removing the cap, be careful not to lose the cap and seal.

Precautions for use - Be sure to read this:

Safety precautions are divided into the following sections, according to the level of danger involved.

⚠ WARNING: Indicates a potentially hazardous situation, which may result in serious personal injuries if not avoided.

⚠ CAUTION: Indicates a potentially hazardous situation, which may result in minor personal injuries and/or property damage if not avoided.

⚠ WARNING

- To prevent abnormalities or worsening of symptoms, people who suffer from the following conditions should refrain from using the product:
 - Those predisposed to allergies or with sensitive skin.
 - * This product contains natural rubber. Natural rubber contains latex allergens and may in certain rare cases cause allergic symptoms such as itching, redness, hives, swelling, fever, difficulty breathing, asthma-like symptoms, a drop in blood pressure, or shock. If any of the above-mentioned symptoms are observed, immediately stop using this product and seek medical attention.
 - Those who have cuts, numbness, eczema, rashes or other abnormalities in the area where the product will be worn.
 - Those who have Raynaud disease (syndrome), other blood convulsive disorders, oversensitivity to cold, poor local blood circulation, or neurological disorders.
- If the product is not used correctly, it may cause circulatory problems, frostbite, or injury. Be sure to follow the warnings below and use the product correctly:
 - The product should only be used for cooling purposes.
 - Do not put anything other than ice or water (hot water, dry ice, or any fluid other than water) into the ice bag.
 - Do not use the product while sleeping.
 - Be careful not to attach the ice bag for prolonged periods, otherwise circulatory problems or frostbite may result.
 - If the product shows any signs of wear or damage, do not use it.
 - Do not attempt to modify or repair the product.
- If any of the following abnormalities occur when using this product, immediately stop using it, and consult a physician. Continued use may worsen symptoms:
 - If abnormalities such as cuts, numbness, eczema, or rashes develop in the area where the product is worn.

⚠ CAUTION

- Friction and use of the product while it is wet may cause the color of this product to fade or be transferred to other materials.
- Be sure to close the cap tightly after putting ice or water into the bag, otherwise the cap may come loose during use, which may result in water or ice leaking out, which may cause your clothing or body to become wet.
- Condensation may cause clothing or the area being cooled to become wet.

Care and cleaning

⚠ CAUTION


- Do not wash this product with light colored items. By doing so, color may fade or be transferred to other materials.

- The product is hand washable. Gently press wash and pat dry using a towel, and line dry in the shade.
- Do not use an iron, chlorine bleach, or oxygen bleach. Doing so may damage the product.
- Do not tumble dry. Doing so may damage the product.
- This product cannot be dry or wet cleaned.

How to use


- Before using this product, carefully read the "Precautions for use - Be sure to read this" section. Then follow the steps provided to ensure the product is correctly used.

1. Fill the ice bag 3/4 full with ice. Adding some water to the bag will help it conform to the shape of your body.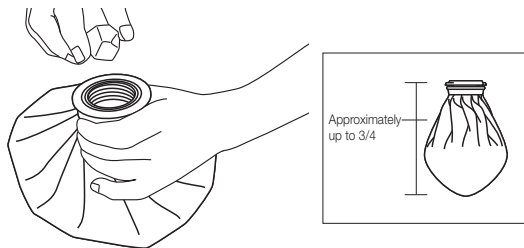

* Putting ice that has sharp edges into the ice bag may damage the inner coating of the bag, causing water to leak out of the bag. Crush ice or add some water to smoothen any sharp edges before putting it into the bag.


2. Let as much air out as possible before screwing the cap on tightly.

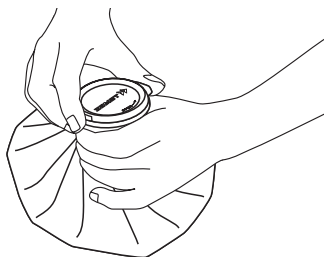3. Apply the ice bag to the desired area.

4. You may apply bandages over the ice bag to create compression and hold the bag in place.

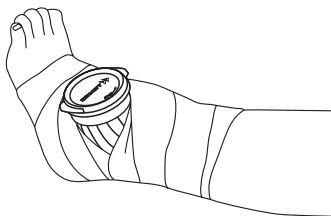* If you use bandages to hold the ice bag in place, do not wrap the bandages excessively tight. Doing so may cause circulatory problems or frostbite.

Disclaimer:

- * The product specifications and appearance are subject to change without notice for the purpose of improvement.
- * When disposing of this product, be sure to follow all disposal ordinances and regulations as specified by the local government.
- * We take all possible measures to ensure product quality, however, if you find any defects, etc. in our products, please contact Nippon Sigmax Co., Ltd.

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